Parsons Grab Bar Installation

Parsons ADL Grab Bars have been designed, engineered and manufactured to provide safety in your bathroom.

****** It is recommended that this product be installed by a qualified or experienced individual to ensure proper and secure installation. Obligation of the supplier and the manufacturer is limited to performance **ONLY** when the bar has been installed properly. No other obligations are assumed by the supplier or the manufacturer.

INSTALLATION:

1. Locate the wall studs where the grab bar will be installed and mark the centerline of each stud.
NOTE: Grab bars must be anchored into wall studs to safely support body weight. Rust resistant screws are included. Designed for use with 3/16" diameter fasteners.
2. Hold the grab bar against the wall at the desired height and angle.
3. Align the mounting flange holes with the stud centerline marks. Use a level to ensure the grab bar is straight.
4. Mark the locations of the mounting holes on the wall through the flange.
NOTE: At least 2 of the 3 mounting holes must be used when mounting horizontally or angled, to ensure the bar stays attached to the wall.
5. Remove the grab bar from the wall and drill pilot holes at the marked locations using an appropriate drill bit. Drill deep enough to penetrate the stud.
6. Place the grab bar back against the wall, aligning the flange holes with the pilot holes. Insert the screws through the appropriate holes in the mounting flange and tighten them securely.
7. For the second side of the grab bar, repeat the marking, drilling and fastening steps above.
8. Ensure the grab bar is level and all screws are firmly tightened. Check that it does not move or loosen when pressure is applied.

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KEGO Corporation